

How to Use Your Homeward Bound Pet[®] Microchip Scanner

1. Insert 9 volt battery, press button to turn on. One beep indicates it is working.
2. Pass over animal's back where shoulders meet. Unit will beep twice when chip is detected.
3. Chip number appears on screen.

Tip: Move the scanner slowly. If chip is not detected, slowly scan down each shoulder and make multiple passes across back. Some microchip brands can migrate after insertion.

To upload chip numbers to your computer:

1. Go to UnitedRegistry.com > Microchips and Scanners. Find your scanner and click to view it. In the product description area, select **Download Driver** and **Download Reader**. It takes just a minute to get them onto your computer.
2. Connect the scanner to your computer with the USB cable provided (port on right side of scanner). Click on the Scanner Uploader icon to open. Select "Link", then "Upload Code."
3. To save the numbers, select "Save (as .txt)." This puts them into a txt file. From here, you can copy and paste them into any spreadsheet or kennel management program on your computer.
4. To clear the numbers on your scanner, select "Clear All." Select "Exit" to quit the program.



Features

- ✓ Detects 125kHz & 134kHz ISO chips (un-encrypted)
- ✓ Accurate in temps from 15 to 120 ° F
- ✓ LCD backlit display
- ✓ Uploads to your PC for record-keeping
- ✓ Low battery alarm
- ✓ Auto shut-off

Questions? Call us toll-free 1-888-908-9299
weekdays 9-5